# AIMS

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2

Organization import					Device basic information import					Sales record import						
Customer name (required)	Person in charge (required)	Contact number	Email	Address	Model name (required)	Serial No (required)	MFG (required)	Purchase	Shipping	Model name (required)	Serial No. (required)	Warranty (required)	Distributor	Customer (required)	Contact (required)	Phone
Lux Hotel	Wang	0225060000	aimsuser1@	No. 221, Sec. 2, Zhishan Rd., Shilin	ZOLL_AED Plus	X13L639168				ZOLL_AED Plus	X13L639168			Lux Hotel	Wang	022506
YangYang Pet	Lin	037970000	aimsuser1@	No. 10, Wenzhong Rd., Taoyuan	Philips_HS1	A13J-05713				Philips_HS1	A13J-05713			YangYang Pet	Lin	037970
Fish ABC	Fiona	0228940000	aimsuser1@	No. 92, Jian 1st Rd., Zhonghe Dist.,	Philips_FRx	B13F-02655				Philips_FRx	B13F-02655			Fish ABC	Fiona	022894

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Customer name (required)	Person in charge (required)	Contact number	Email	Address
Lux Hotel	Wang	0225060000	aimsuser10	No. 221, Sec. 2, Zhishan Rd., Shilin Dist., Taipei City, Taiwan
YangYang Pet	Lin	037970000	aimsuser10	No. 10, Wenzhong Rd., Taoyuan Dist., Taoyuan City, Taiwan
Fish ABC	Fiona	0228940000	aimsuser10	No. 92, Jian 1st Rd., Zhonghe Dist., New Taipei City, Taiwan

A	B	C	D	E
公司名称 (required)	联络人 (required)	联系电话	邮箱	地址
Lux Hotel	Wang	0225060000	aimsuser10	No. 221, Sec. 2, Zhishan Rd., Shilin Dist., Taipei City, Taiwan
YangYang Pet	Lin	037970000	aimsuser10	No. 10, Wenzhong Rd., Taoyuan Dist., Taoyuan City, Taiwan
Fish ABC	Fiona	0228940000	aimsuser10	No. 92, Jian 1st Rd., Zhonghe Dist., New Taipei City, Taiwan

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The screenshot shows the AED Cloud Management Platform interface. The main navigation menu on the left includes: 首页, 我的消息, 销售管理, 监测, 工单, 报告, 统计分析, 帐号管理, and 组织管理. The current page is '组织管理' (Organization Management). A modal dialog box titled '导入' (Import) is open, showing '组织类型' (Organization Type) with radio buttons for '代理商' (Agent), '部门' (Department), and '客户' (Customer). A dropdown menu for '\* 上级组织' (Superior Organization) is highlighted with a red box and the number 8. Below the dialog, a file selection window is open, showing a list of files. The file 'Organization import template' is highlighted with a red box and the number 9. A red text box with the text '组织信息导入' (Import Organization Information) is overlaid on the file selection window.

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The screenshot shows the AED Cloud Management Platform interface. The left sidebar contains a navigation menu with the following items: 首页 (Home), 我的消息 (My Messages), 销售管理 (Sales Management), 库存管理 (Inventory Management), 销售纪录 (Sales Record), 监测 (Monitoring), 工单 (Work Order), 报告 (Report), 统计分析 (Statistical Analysis), and 账号管理 (Account Management). The '销售管理' (Sales Management) menu item is highlighted with an orange box and a circled '1'. Below it, the '型号' (Model) sub-menu item is also highlighted with an orange box and a circled '2'. The main content area shows a table of AED models with columns for '操作' (Action), '品牌' (Brand), '型号' (Model), '产品名称' (Product Name), and '状态' (Status). The table contains 10 rows of data, including models from PHILIPS, defibtech, ZOLL, NIHON KOHDEN, and HEARTSINE. A circled '3' is placed over the first row of the table. The table data is as follows:

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The screenshot shows the AED Cloud Management Platform interface. The breadcrumb navigation is '首页 / 销售管理 / 库存管理'. The left sidebar has '销售管理' (Sales Management) highlighted with a red box and '1', and '库存管理' (Inventory Management) highlighted with a red box and '2'. The main content area has a '+ 新增' (Add) button highlighted with a red box and '3'. Below the buttons is a table with columns: 操作, 单位, 产品序号, 生产日期, 购入日期, 出库日期, 出售状态. The table is currently empty, showing '暂无数据' (No data).

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The screenshot shows the '工单总表' (Work Order Summary) page. A sidebar on the left contains a '新增' (Add) button, which is circled in orange and labeled with a '9'. Below the button are four options: '报警工单' (Alarm Work Order), '保养工单' (Maintenance Work Order), '培训工单' (Training Work Order), and '其他工单' (Other Work Order). An arrow points from the '保养工单' option to a modal window titled '新增保养单' (Add Maintenance Order), which is also circled in orange and labeled with a '10'. The modal window contains the following fields:

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The '新增保养单' form includes the following fields:

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At the bottom of the modal are '确定' (Confirm) and '取消' (Cancel) buttons.

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2

Organization import					Device basic information import					Sales record import						
Customer name (required)	Person in charge (required)	Contact number	Email	Address	Model name (required)	Serial No (required)	MFG (required)	Purchase	Shipping	Model name (required)	Serial No. (required)	Warranty (required)	Distributor	Customer (required)	Contact (required)	Phone
Lux Hotel	Wang	0225060000	aimsuser1	(No. 221, Sec. 2, Z	ZOLL_AED Plus	X13L639168				ZOLL_AED Plus	X13L639168			Lux Hotel	Wang	022506
YangYang Pet	Lin	037970000	aimsuser1	(No. 10, Wenzhon	Philips_HS1	A13J-05713				Philips_HS1	A13J-05713			YangYang Pet	Lin	037970
Fish ABC	Fiona	0228940000	aimsuser1	(No. 92, Jian 1st R	Philips_FRx	B13F-02655				Philips_FRx	B13F-02655			Fish ABC	Fiona	022894

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The screenshot shows the AIMS inventory management interface. The 'Import' button is highlighted with a red circle and the number 7. A file explorer window is open, showing a list of files. The file 'Device basic information import template' is highlighted with a red circle and the number 8. A red text box with the text '设备基本信息导入' (Import Device Basic Information) is overlaid on the file explorer window.

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4 Sales record import

Model name (required)	Serial No. (required)	Warranty (required)	Distributor	Customer (required)	Contact (required)	Phone	Job title	Email	Address (required)	Location	Installation end time	License administrator	Training Program	Person to be notified (required)	Work order processor (required)
ZOLL_AED Plus	X13L168168	2020/4/25		Lux Hotel	Wang	0225060000		aimsus	No. 221, Sec. 2, Zhisha	Lobby	2015/4/25			global	global_op
Philips_HS1	A13J-07777	2019/4/9		YangYang Pet	Lin	037970000		aimsus	No. 10, Wenzhong Rd.	Counter	2014/4/9			global	global_op
Philips_FRx	B13F-05555	2019/4/22		Fish ABC	Fiona	0228940000		aimsus	No. 92, Jian 1st Rd., Zh	Counter	2014/4/22			global	global_op

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⑤ 导入AIMS。

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The screenshot shows the AED Cloud Management Platform interface. The sidebar on the left contains navigation options: 首页 (Home), 我的消息 (My Messages), 销售管理 (Sales Management), 监测 (Monitoring), 工单 (Work Order), 报告 (Report), 检验报告 (Test Report), and 统计分析 (Statistical Analysis). The 'Report' and 'Test Report' options are highlighted with orange boxes and numbered 1 and 2 respectively. The main content area displays a table of test results with columns for 操作 (Action), 单位 (Unit), 状态 (Status), 产品类别 (Product Category), 产品名称 (Product Name), 产品序号 (Product Serial No.), and 报告编码 (Report Code). A row of data is highlighted with an orange box and numbered 3, showing a status of 'To be submitted'. Below the table, there are pagination controls showing 'Total 1' and '10/page'. A confirmation dialog box is overlaid on the bottom right, asking '确定提交检验报告?' (Confirm submitting test report?) with '取消' (Cancel) and '确定' (Confirm) buttons.

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# Test Solutions **for**

Medical Device Manufacturers

[www.whaleteq.com](http://www.whaleteq.com)  
[service@whaleteq.com](mailto:service@whaleteq.com)

 8F., No. 125 Songjiang Rd., Zhongshan Dist., Taipei City 104474, Taiwan

 +886-2-2517-6255

 +886-2-2596-0702

Copyright © 2013-2023, All Rights Reserved.  
WhaleTeq Co. LTD

is a registered trademark of WhaleTeq Co. LTD.

All other trademarks or trade names are the property of their respective holders.

All information, documentation, firmware, software utilities, and specifications are subject to change without prior notification by the manufacturer.