



# DFS360

## AED / Defibrillator Tester

Displays defibrillation shock waveform, energy (in Joules), parameters, and characteristic points at the same time

- ♦ Intuitive software interface with shock-related information on one screen – the waveform, energy (in Joules), parameters, and characteristic points captured at a sampling rate of 4μs
- ♦ Automatically saves complete raw data of the shock waveform to establish a test history
- ♦ Outputs ECG waveforms with selectable adult and child modes to achieve high accuracy for testing
- ♦ Raw data playback – plays recorded raw data or customized waveforms for repeated verification to facilitate the development process
- ♦ Repeatability and Reproducibility Analysis – compares the shock waveforms of multiple DUTs or a single DUT to confirm the consistency of quality and reliability, facilitating conducting the endurance test described in IEC 60601-2-4\*
- ♦ Provides software development kit (SDK) for developing automated testing programs

\*Note: The Repeatability and Reproducibility Analysis needs a separate purchase.

## Specifications

### Energy Measurement Specifications

Items	Specifications
Maximum Energy	Range: Up to 600J Accuracy: $\pm 1\%$ of reading $\pm 1J$ for up to 360J; $\pm 1\%$ of reading $\pm 1J$ , typical for 360J ~ 600J
Voltage	Range: Up to 3000V Accuracy: $\pm (1\% \text{ of reading} + 1V)$ for 450V ~ 3000V & -450V ~ -3000V; $\pm (1\% \text{ of reading} + 2V)$ for -450V ~ 450V
Current	Range: 1.2 ~ 100.0A Accuracy: $\pm (1\% \text{ of reading} + 0.1A)$
Load Resistance	Resistance: 50 $\Omega$
Energy Accuracy	For 50 $\Omega$ : $\pm 1\%$ of reading $\pm 0.1J$ for up to 360J; $\pm 1\%$ of reading $\pm 0.1J$ , typical for 360J ~ 600J Note: Maximum energy of monophasic defibrillation may not reach 360J.
Pulse Width	Range: 1.0 ~ 60.0ms Accuracy: $\pm 0.05ms$
Sample Rate	250kHz (4 $\mu s$ sample)
Maximum Average Power	12W, equivalent to 1 defibrillation pulse of 360J every 30 seconds

### General Specifications

Items	Specifications
Temperature	Operating: 0°C ~ 50°C (32°F ~ 122°F) Storage: -20°C ~ +60°C (-4°F ~ +140°F)
Humidity	10% ~ 90% non-condensing
Communications	USB Type-C connector
Operation Modes	Remote (PC Software via API and SDK)
Power	USB 3.0 Bus Power
Mechanical	Housing: ABS Plastic Size (L x W x H): 19.0 x 12.5 x 6.0cm Weight: 665g

## Ordering Information

### DFS360 Test System

Part No.	Description	Quantity
100-DF00003	Model No.: DFS360 Model Name: AED/Defibrillator Tester	1

### Optional Accessories

Part No.	Description	Quantity
K22-0500101	Open-ended AED Cable (Banana Male), 0.5 meter (Black)	1
K22-0500102	Open-ended AED Cable (Banana Male), 0.5 meter (Red)	1
N61-0210033	Compact Splicing Connector	1
K27-1800304	USB Transfer Cable: Type-A to Type-C (Male to Male), 1.8 meter	1

### Optional Paddle Box Set

Part No.	Description	Quantity
300-DFPD0CK	DFS Tester Docking Base for Defibrillator	1
E13-070P101	DFS Tester Defibrillation Receiving Plate	2
K22-0100301	AED Cable for Paddle Box, Female to Male (Black)	1
K22-0100201	AED Cable for Paddle Box, Female to Male (Red)	1

### Optional Tool

Part No.	Description	Quantity
800-DF07001	Scan Software and Barcode Scanner (Argox barcode scanner, model no. AS-9400)	1

### Optional Software Add-On Pack

Part No.	Description
HE0-DF00001	Repeatability and Reproducibility Analysis for inspecting DUT's quality and performance

### Optional Calibration Service and Warranty Extension

Part No.	Description
YY0007	Model No.: C3 Provides (3) years of calibration service coverage. WhaleTeq equipment can be calibrated to original performance on the basis of (1) year interval.
YY0008	Model No.: R3 Extends the limited warranty from (1) year to (3) years.

### ECG Signal Specifications


Items	Specifications
Ventricular Fibrillation - Coarse	Vp-p = 3mV
Ventricular Fibrillation - Fine	Vp-p = 0.5mV
Ventricular Tachycardia ( $\approx 180BPM$ )	Vp-p = 3mV
Ventricular Tachycardia Plus ( $\approx 234BPM$ )	Vp-p = 3mV
Atrial Fibrillation	Vp-p = 1.2mV
Asystole	Vp-p = 0.03mV
Normal Sinus Rhythm (NSR) (20 ~ 360BPM by 1BPM per step)	Vp-p 0.5 ~ 5.0mV, step 0.5mV
User-defined Pattern (Max. 5kHz Sample Rate)	Load Customized Waveform function: Load a total capacity of up to 1000 seconds of raw data files with variable duration for each file to the DFS360 memory. Play Data function: Directly select and play raw data files stored on the computer with no file size limitations.
RCA Output Amplitude Range	1:880
RCA Output Amplitude Accuracy	$\pm 5\%$ Vpp

### Compliance Specifications


Items	Specifications
Safety Standards	CE: IEC / EN61010-1 : 2010 + A1 : 2016; Pollution degree 2 61010-2-030 : 2017
Electromagnetic Compatibility Standards (EMC)	CE: EN61326-1 FCC: EMC P15B

\* Specifications are subject to change without prior notice.

### • AED Testing Scenario

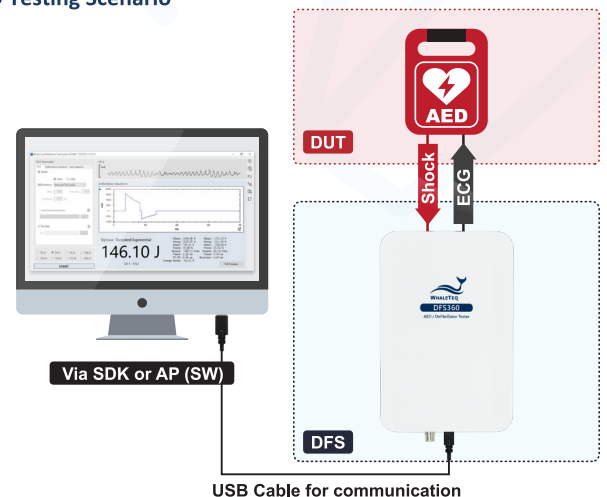

### • Defibrillator Testing Scenario (Optional)

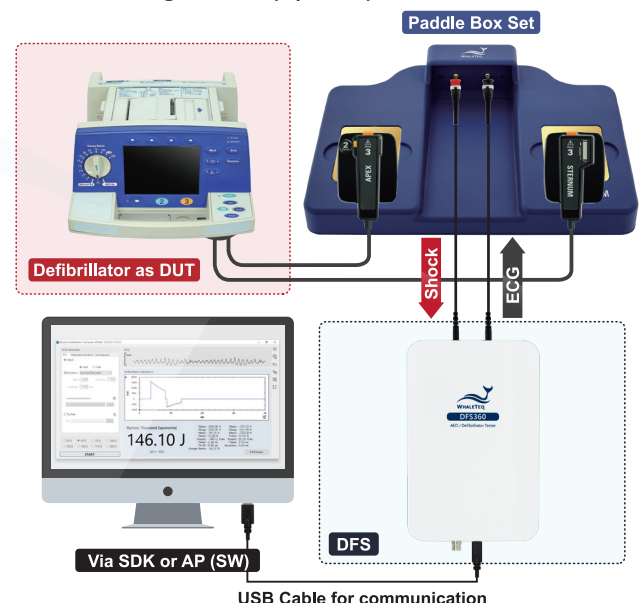

**DFS360**  
Scan for  
more information

[www.whaleteq.com](http://www.whaleteq.com)

[service@whaleteq.com](mailto:service@whaleteq.com)

8F., No. 125 Songjiang Rd., Zhongshan Dist., Taipei City 104474, Taiwan

+886-2-2517-6255

+886-2-2596-0702