



## WhaleTeq HRS200 SDK

Version: 1.0.4 Build: 20191107

=====

1. Support "x64" platform.
2. Support "without UI Windows" usage like Python Script or Console Program.
3. Fix "Enum OutputFunction\_E { Output\_PPG = 14 } error, the number should be 15."

Version: 1.0.3 Build: 20170929

=====

1. Add new boolean function for checking whether the wider PD range is supported on connected HRS device.
2. Add new API for PD Sampling Channel setting - "void SetSampleChannel(int Channel)".

Version: 1.0.2 Build: 20170111

=====

Support new demo package "WhaleteqPPG\_SDK\_FineTune" for "Fine Tune" related APIs.

Version: 1.0.1 Build: 20161102

=====

Support "StartPPG()" without connCB parameter for LabView.  
Fix "RegisterPDSamplingCB()" bug.

Version: 1.0.0 Build: 20160805

=====

First version release